

**THE 3-IN-1 SYSTEM**  
THAT REDEFINES THE RULES OF FACIAL TREATMENTS



**Aqua3**  
TECH LINE

**WE BELIEVE  
EVERY WOMAN DESERVES  
MORE THAN SIMPLE  
FACIAL AESTHETIC TREATMENTS.**



**AQUA3** is a **3-in-1** technology that combines the extraordinary detergent properties of **water-based peel containing sequential acids** with microdermabrasion and the **transdermal delivery** of cosmetic ingredients.

**This is an extremely effective system that raises the bar for any facial treatment,** taking the effective properties of active substances to a new level.

This is why we have created an extremely effective method that raises the bar for any facial treatment, taking the effective properties of active substances to a new level.



**IT CLEANSSES  
YOUR FACE DEEPLY**  
thanks to the water-based peel



**IT REMOVES  
DEAD SKIN CELLS**  
to obtain a more effective exfoliation



**IT REBALANCES SEBUM AND  
MAKES COMEDONES DISAPPEAR**  
with the power of vacuum



**IT MOISTURISES  
YOUR SKIN DEEPLY**  
thanks to the infuser handpiece



**IT RESTRUCTURES  
YOUR TISSUES**  
with transdermal delivery



**DIAMOND  
MICRODERMABRASION**  
it is effective on thickened skins,  
acne scars, deep wrinkles  
or stretch marks.



## AQUA3 HANDPIECES

A new hydrodermabrasion treatment that combines in one technology **cleansing, moisturising and rejuvenation.**



### INFUSER/ASPIRATING HANDPIECE

The water-based peel containing sequential acids **allows you to deep cleanse your face and obtain a long-lasting moisturising effect.**



### MESOPORATION HANDPIECE

Through the transdermal delivery of active substances, **this handpiece makes any treatment extremely effective** and allows you to restructure your skin from within.



### DIAMOND-GRAIN HANDPIECE

The properties of diamond grains **allow exfoliating and renovating any type of skin,** including thickened skins, skins with deep wrinkles, acne scars or stretch marks.

NEW 




## HOW AQUA3 WORKS

**Cleansing your skin only with cleansing milk and tonic is not enough** to prepare it for treatment.

The outermost stratum corneum, which thickens over the years, makes the skin less bright and soft and creates a barrier that prevents active substances to penetrate.

### 1 WATER-BASED PEEL WITH SEQUENTIAL ACIDS

The aspirating handpiece, together with a water jet to which **Mandelic Acid** and **Salicylic Acid** have been added, removes superficial impurities even from thickened skins by aspirating dead skin cells, blackheads and whiteheads thanks to the powerful effect of vacuum.

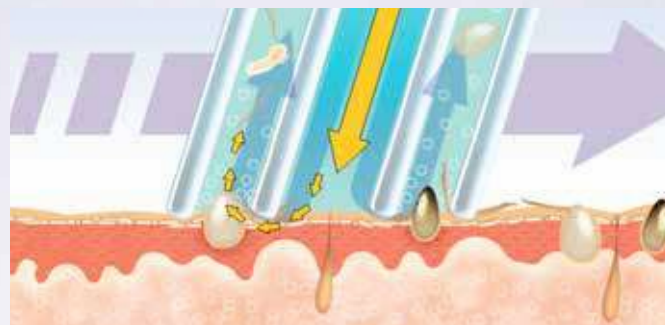

### 2 DEEP MOISTURISATION WITH HYALURONIC ACID

By using the infuser handpiece with low-molecular-weight **Hyaluronic Acid**, your skin is deeply moisturised and appears as glowing and smooth as never before.



### 3 RESTRUCTURING AND OXYGENATING YOUR TISSUES

By making the most out of the transdermic delivery of the mesoporative handpiece, your skin is restructured by the **Hyaluronic Acid** and Dermakléb serums.

## DIAMOND-GRAIN HANDPIECE

The diamond-grain handpiece is a new device to treat the most critical areas, like the T-zone, in a selective and delicate manner, while at the same time reducing the appearance of scars and stretch marks by penetrating deep.

**9 different heads**, various levels of abrasive power, for an extra-exfoliating effect and amazing results.

#### LIGHT ACTION



for small scars, fine lines, the area around the eyes, the nose and the lips, small dischromic patches

#### MEDIUM ACTION



for spots and medium-large dischromic areas, deep wrinkles, and blackheads

#### STRONG ACTION



for acne, stretch marks, scars, knees, elbows and thickened areas of your body



### THE PROPERTIES OF DIAMOND TO RENOVATE EVERY TYPE OF SKIN

An ordinary water-based peel is not enough to treat largely thickened skins. With its unique diamond-grain handpiece, AQUA3 allows treating the most problematic skins by performing a controlled, non-invasive microdermabrasion that can remove the outermost stratum corneum, thus improving skin texture and restoring skin elasticity.

ALSO  
FOR YOUR BODY

## AQUA3TREATMENT

In order to obtain the **best results**, we have created a **special protocol**, that redefines the rules of facial treatments by providing a holistic effect.

### SKIN CLEANSING

Once make-up has been removed with the most suitable Dermakléb, a water-based peel containing sequential acids is performed. The intensity of the peel can be adjusted and it is customisable according to the type of skin to be treated.

#### STEP 1

A beauty therapist sweeps the aspirating handpiece over your face. Thanks to a pleasant water jet to which **Mandelic Acid** has been added, it removes superficial impurities in a delicate but firm manner.

#### STEP 2

The following step is a water-based peel containing **Salicylic Acid** to perform a deeper peel on thickened skins.

#### STEP 3

The appropriate **Dermakléb Peel** is applied on your face for 5 minutes. Once removed, all the dead skin cells are aspirated.

### MOISTURISATION

#### STEP 4

Your skin gets deeply moisturised thanks to the infuser handpiece containing low-molecular-weight **Hyaluronic Acid**.



### RESTRUCTURING

#### STEP 5

A thin layer of Dermakléb serum is applied on your skin with the mesoporation handpiece.



### OXYGENATING

#### STEP 6

The Dermakléb active oxygen **OxyPlus mask** is applied on your face, neck and décolletage. In a matter of seconds, it will turn into a soft, creamy mousse, which after 2-3 minutes can be rubbed all over your face before being removed.



THE TREATMENT MUST BE REPEATED EVERY WEEK FOR 4 WEEKS.  
THE MAINTENANCE TREATMENT REQUIRES 1 SESSION A MONTH.

Depending on the type of skin and degree of skin aging, some of the steps may be skipped.



## PROFESSIONAL PRODUCTS

The exfoliation performed with unique diamond-grain heads and the application and delivery of multi-functional serums, which penetrate deep thanks to the infuser handpiece, allow you to get **incredible results right from the first session.**

The deeply rejuvenating treatment is complemented by the OxyPlus mask: **a real regenerating “bath”** that, thanks to the thousands of active oxygen bubbles, **gives your skin a flawless, healthy and glowing look .**



The image portraying the foam created by the active oxygen mask is for illustrative purposes only. The product tone and consistency may vary.



### MANDELIC ACID

Mandelic Acid-based solution for an extra-delicate peel.

30 ml flask



### SALICYLIC ACID

Salicylic Acid-based solution for a gentle, yet intense peel at the same time.

30 ml flask



### HYALURONIC ACID

Hyaluronic Acid-based solution low-molecular-weight to obtain deep, long-lasting moisturisation.

30 ml flask



### ACTIVE OXYGEN OXYPLUS MASK

It gives your skin an “oxygen bath” that **eliminates toxins and moisturises deeply**, a real regenerating treatment repair treatment. Thousands of bubbles created inside the liposome core act as magnets and attract oxygen. Once they are activated by a gentle massage, they create a soft mousse, rich in **active substances that can boost oxygenation and cell metabolism**, thus reinvigorating worn-out and tired skin, by nourishing it deeply.

100 ml airless flask

All the acids must be diluted with water in the appropriate 500 ml AQUA3 flask

## HOME PRODUCTS

All the properties of cosmeceuticals are enshrined in just one capsule.

The **Dermakléb SkinCaps** contain a high concentration of the purest active and functional substances. By leveraging the full potential of modern dermocosmetic technology, these substances preserve the integrity of the encapsulated cosmetic until it is released.

The **Dermakléb SkinCaps** come in daily doses and are stored in special dispenser containing **28 treatments** to take with you wherever you go.

DAY HOME **28** PROGRAMME

## THE FIRST INJECTION-FREE FILLER WITH DOUBLE COSMETIC ACTION FOR A SKIN RENOVATED IN JUST 28 DAYS

Thanks to the instantaneous effect of Hyaluronic Acid and Biopeptides extracted from yeast, the **Dermakléb SkinCaps reduce deep wrinkles for a long-term effect**

- 1 REDUCTION OF WRINKLES**  
with immediate effect
- 2 +20% COLLAGEN\***  
and elastin for a firmer skin
- 3 ANTI-DARK CIRCLE ACTION**  
thanks to oligopeptide microspheres
- 4 BRIGHTER SKIN**  
with Hyaluronic Acid microspheres

### REDUCTION OF WRINKLES THANKS TO THE IMMEDIATE EFFECT

The **hyaluronic acid microspheres** and the **Biopeptides extracted from yeast** trigger a quick moisturising process for a visible **reduction of deep wrinkles**.

Results in 1-2 hours after application

Reduction of wrinkles and fine lines	80%**
Glowing skin	80%**
Smooth skin	95%**

Results right after the first application

Reduction of dark circles	95%**
Less visible wrinkles	90%**
Reduction of eye bags	93%**

### NIGHT ANTI-DARK CIRCLES ACTION

**Vitamin A** microspheres stimulate skin renewal, while **Chrysin** reduces livid circles. **Dark circles appear visibly reduced** in only 15 days after application.

\*Average increase in the number of fibroblasts responsible for the synthesis of new collagen and elastin.

\*\*Instrumental tests carried out on 20 subjects. Percentage referred to the number of subjects who underwent this treatment, declaring the above-mentioned benefits.

## HOME PRODUCTS

The **Vegicaps patented technology**<sup>®</sup> of SkinCaps monodose ensures the maximum effectiveness and hygiene of the product without adding any paraben, excipient, perfume or dye.



WEEK 1:  
apply 1 skincaps in the morning  
and 1 at night

WEEK 2:  
apply 1 skincaps a day  
preferably in the morning.

WEEK 3 AND 4:  
apply 1 skincaps every other day,  
preferably in the morning.

### DERMAKLÉB SKINCAPS DOUBLE EFFECT

ANTI-AGING, MOISTURISING, FIRMING EFFECT

**A 28-day treatment with Hyaluronic Acid and Biopeptides extracted from yeast** that helps you reduce first and second-degree wrinkles, thus triggering collagen and elastin production.

Your skin immediately looks more compact, relaxed and glowin  
Dispenser containing 28 0.6 ml capsule

DAY TREATMENT



### DERMAKLÉB SKINCAPS EYES LIFT

ANTI-AGING, DARK CIRCLES AND EYE BAGS

**28-night treatment** that effectively counteracts wrinkles around the eyes as well as dark circles and eye bags, thanks to **Vitamin A**, which boosts skin cellular renovation and **Chrysin**, a flavonoid that reduces purple complexion.

Dispenser containing 28 0.31 ml capsule

NIGHT TREATMENT



APPLY  
EVERY NIGHT  
FOR 28 DAYS



## IN THE HANDY COUNTER DISPLAY

### WITH 32 2-CAPSULE TEST KITS

HANDY, PRACTICAL, WITH AN INNOVATIVE DESIGN  
This counter display has been designed to showcase the two Skincaps dispensers, with 32 2-capsule test kits in order to facilitate sales and allow customers to test how effective the product is.  
Counter display 32 2-Skincaps test kits.

## DERMAKLÉB SILVER MASK

ANTI-AGING WITH MEMORY EFFECT

**If used before going to bed, it fights the signs of aging** as it adapts to your face thanks to the **memory effect**.

The gradual release of **stabilised Vitamic C** and the mineralising properties of **Silver Dag** give firmness to the skin, which looks more glowing and homogeneous.

50 ml jar

APPLY TWICE A WEEK, BEFORE  
GOING TO BED,  
WITH THE HANDY BRUSH


NIGHT TREATMENT






## SKINCAPS DOUBLE EFFECT RESULTS


**Patented immediate effect:** results in 1-2 hours after application

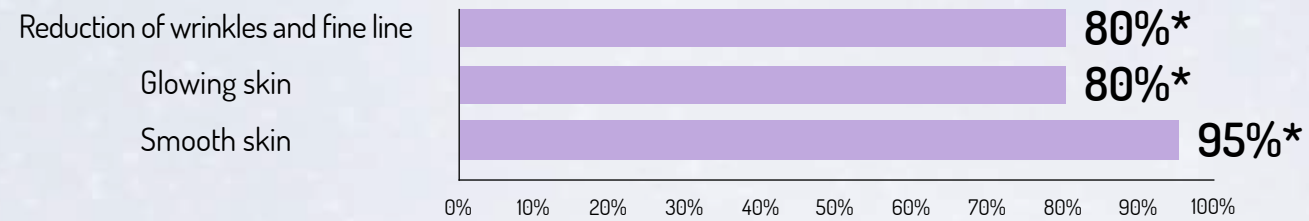

**Long-term effect \*\*:** results in 4 weeks after application

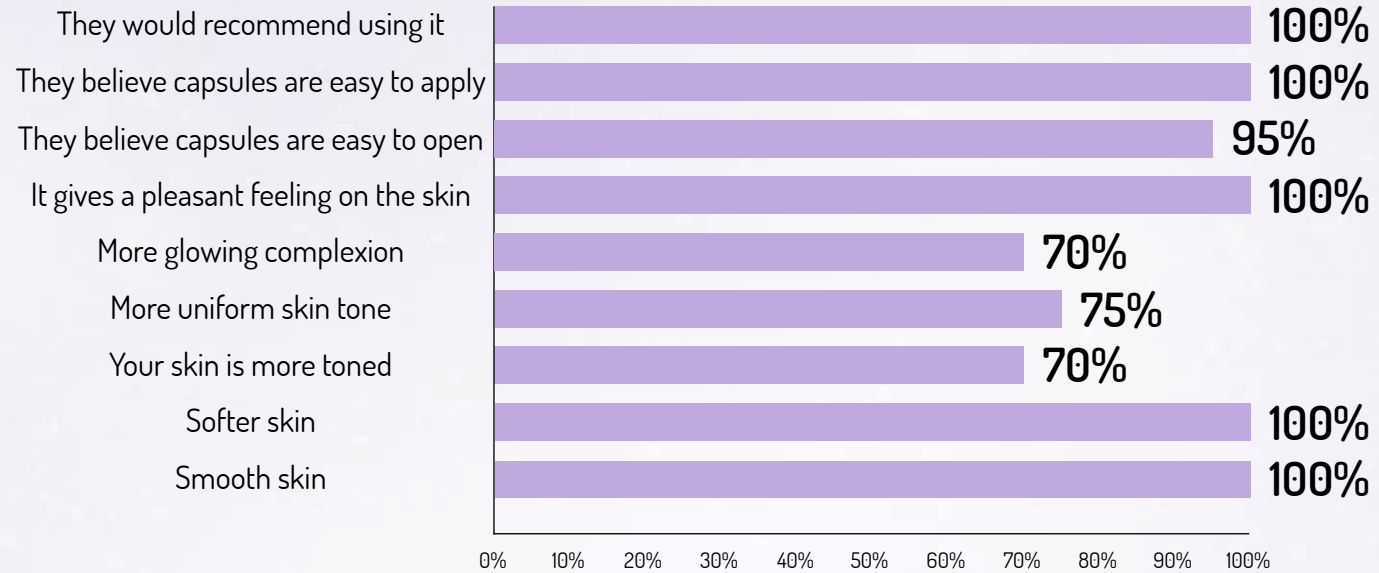

**Reduction of deep wrinkles** compared to their initial condition

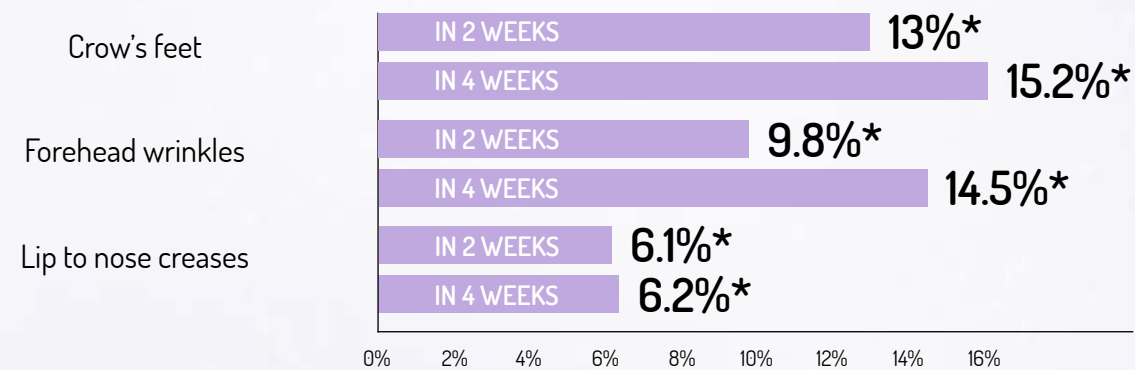


\*Instrumental tests carried out on 20 subjects. Percentage referred to the number of subjects who underwent this treatment, declaring the above-mentioned benefits.


\*\*Self-assessment tests

## SKINCAPS EYES LIFT RESULTS

Using the product is proven to **significantly improve the look of your skin.**

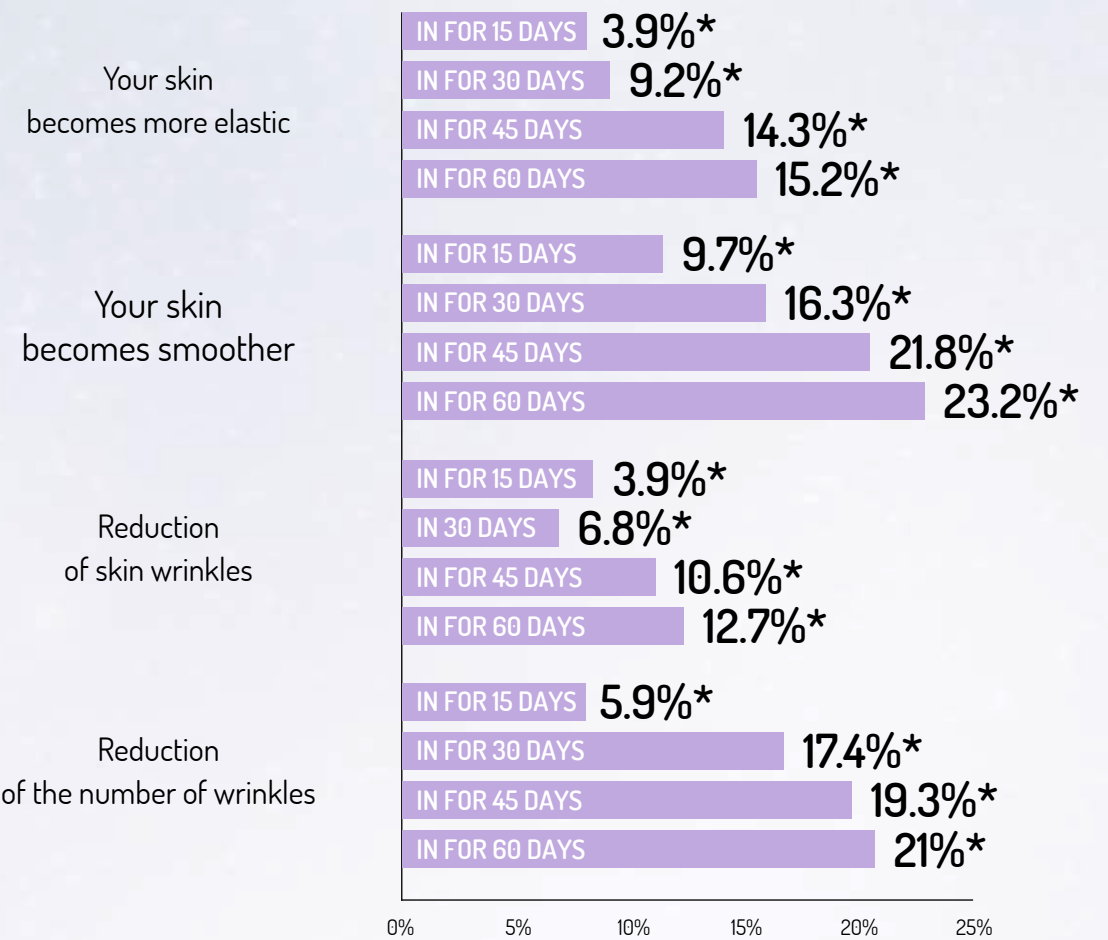

**Reduction of dark circles** blue and red components

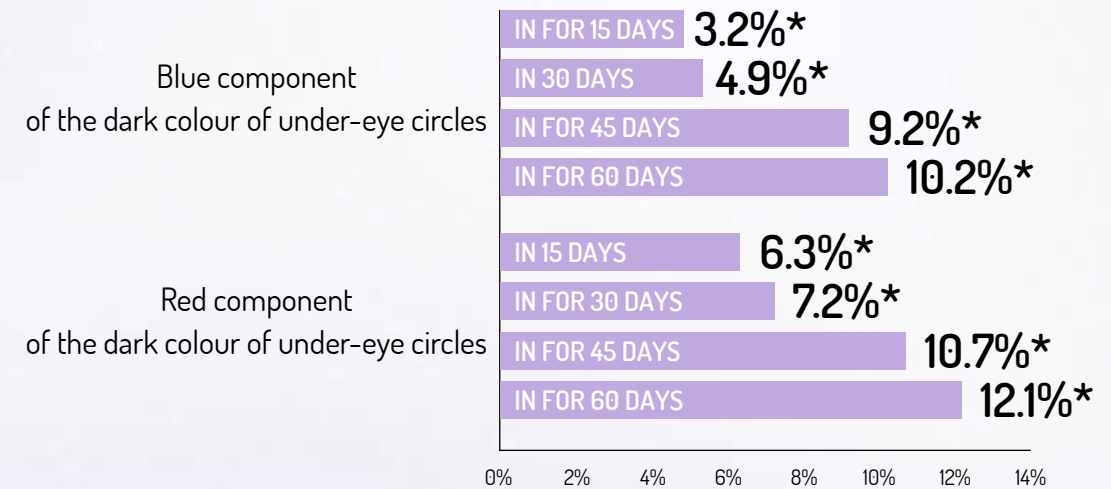

\*instrumental tests performed on 20 subjects

## SUMMARY OF PROFESSIONAL AND HOME-USE PRODUCTS

	PRODUCT	FORMAT	CODE	USE
	MANDELIC ACID	30 ml flask	MAC05/S1	Professional
	SALICYLIC ACID	30 ml flask	MAC05/S2	Professional
	HYALURONIC ACID	30 ml flask	MAC05/S3	Professional
	ACTIVE OXYGEN OXYPLUS MASK	100 ml airless flask	DERM000116	Professional
	DERMAKLÉB SKINCAPS DOUBLE EFFECT	Dispenser containing 28 0.6 ml capsules	DERM000117	Home use
	DERMAKLÉB SKINCAPS EYES LIFT	Dispenser containing 28 0.31 ml capsules!	DERM000118	Home use
	SKINCAPS SKINCAPS Minikit	Blister x 2 SkinCaps	DERM000119	Home use
	SKINCAPS EYES LIFT Minikit	Blister x 2 SkinCaps	DERM000120	Home use
	SILVER FACE MASK with Colloidal Silver and stabilised Vitamin C	jar + brush	DERM000115	Home use





# Aqua3

 B-TECH LINE

**B A L D A N**  
G R O U P

[www.baldangroup.com](http://www.baldangroup.com)

B&M Srl - Marketing nel Benessere  
Via L. Bruni, 25 - Milano - ITALY

